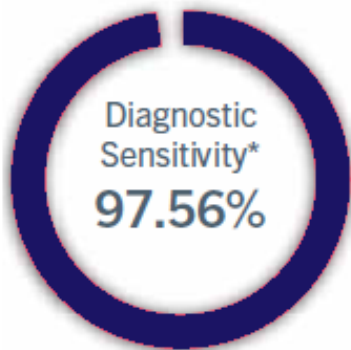
# COVID-19 Ag Rapid Test

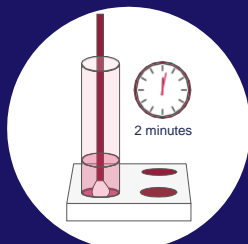
\*ct value: 20 - 30

Rapid test for the detection of SARS-CoV-2 viral nucleoprotein antigens in human nasopharyngeal and oropharyngeal specimens

Sample

Extract

Test

Result


The COVID-19 Rapid Antigen Point Of Care Test

The NADAL<sup>®</sup> COVID-19 Ag Test is a lateral flow chromatographic immunoassay for the qualitative detection of SARS-CoV-2 viral nucleoprotein antigens in human nasopharyngeal and oropharyngeal specimens. This real-time test is a practical solution that is straightforward to implement, even at scale. It is intended for use as an aid in the diagnosis of COVID-19 infections, conducted by a healthcare professional, with results provided directly to patients within 20 minutes with SARSCoV-2.

The NADAL<sup>®</sup> COVID-19 Ag Test detects COVID-19 directly by looking for the presence of specific proteins from the SARS-CoV-2 virus. Thus, antigen detection closes a diagnostic gap. The nucleocapsid protein (N) is ideal for virus detection, as it is highly abundant in the virus and sufficiently specific for SARS-CoV-2. It is one of the 4 most important structural proteins and is involved in replication, transcription and packaging of the viral genome.

The NADAL<sup>®</sup> COVID-19 Ag Test has been evaluated with clinical specimens whose status was confirmed using RT-PCR. The results are presented in the following table:

		RT-PCR		
		Positive	Negative	Total
NADAL <sup>®</sup> COVID-19 Ag Test	Positive	150	0	150
	Negative	37	161	198
	Total	187	161	348

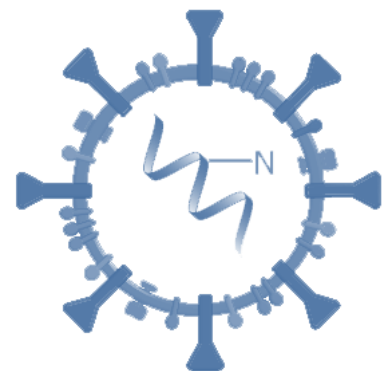
### Benefits at a glance

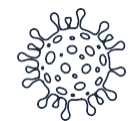- ✓ High specificity
- ✓ High sensitivity with high viral loads
- ✓ Easy to use
- ✓ Fast and reliable test results in only 15 minutes
- ✓ Test can be performed using naso- and oropharyngeal specimens
- ✓ Can be stored at room temperature
- ✓ All test components (including sterile swabs) are included
- ✓ No cross-reactivity with human pathogenic coronaviruses (like nCoV-229E, -HKU1, -NL63 ND -OC43) or influenza viruses (like influenza A/B)

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