

COVID-19 IgG/IgM Rapid Test Kit (Colloidal Gold)



*IgG values


Rapid test for the detection of SARS-CoV-2 antibodies in human blood, plasma or serum samples




Puncture



Collect

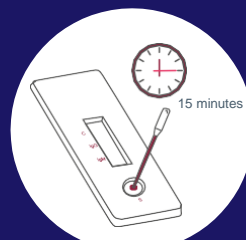

Test

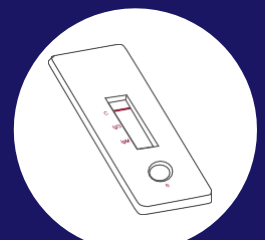

Result


The COVID-19 Rapid Antibody Point Of Care Test

The NADAL® COVID-19 IgM/IgG Test is a lateral flow chromatographic immunoassay for the qualitative detection of anti-SARS-CoV-2 IgG and IgM in human whole blood, serum or plasma specimens of symptomatic patients. This real-time test is a practical solution that is straightforward to implement, even at scale. It is intended for use as an aid in the diagnosis of COVID-19 antibody response, conducted by a healthcare professional, with results provided directly to patients within 15 minutes.

The NADAL® COVID-19 IgM/IgG Test detects human antibodies to the COVID-19 virus. The presence of a red line in the test line region 'IgM' and/or 'IgG' indicates a positive result. The absence of a red line in the test line region 'IgM' and/or 'IgG' indicates a negative result.

The NADAL® COVID-19 Ag Test has been evaluated with clinical specimens whose status was confirmed using RT-PCR. The results are presented in the following table:

		RT-PCR		
		Positive	Negative	Total
NADAL® COVID-19 IgG/IgM Test (IgG)	Positive	82	3	85
	Negative	1	224	225
	Total	83	227	310

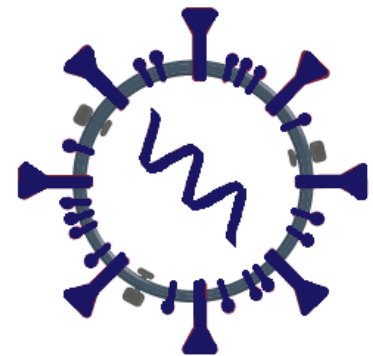

Benefits at a glance

- ✓ High specificity
- ✓ High sensitivity
- ✓ Easy to use
- ✓ Fast and reliable test results in only 15 minutes
- ✓ Test can be performed using blood sample
- ✓ Can be stored at room temperature
- ✓ All test components (including sterile swabs) are included
- ✓ No crossreactivity with the specimens was observed when tested using the NADAL® COVID-19 IgG/IgM Test.




CE
Marked



German
Manufacturer



Immediate
Results

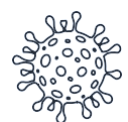

COVID-19
Specific



Deployable
at Scale



Designed
for HCPs

Tel: +96264622301 E-mail: info@bioclincalme.com